

Cordless, disposable Simple to operate Improved safety



The LiNA Xcise™ is a cordless, fully disposable Morcellator intended for tissue morcellation during laparoscopic gynaecological procedures. The unique design provides safe and easy control during hysterectomy and myomectomy.



Adjustable coreguard



Reducer cap



Flared inner tube

The Xcise Morcellator focuses on providing you with a simple and efficient solution for morcellation. The protective cannula serves as a safety guard and ensures a secure introduction into the abdomen, while the flared design of the cutting tube minimizes tissue rotation during morcellation.

The Xcise Morcellator can be used straight from the packaging, no capital equipment or additional investment is needed.



Simplicity

Single use device that is completely ready to use

- Cordless and no assembly required
The morcellator can be used right out of the box and as it is cordless there is no need for assembly to driver box or cable. The LiNA Xcise™ is therefore quick, simple and easy to use and facilitates unlimited mobility of the OR staff.
- Fully disposable
The LiNA Xcise™ is RoSH compliant and fully disposable in regular hospital waste.
- Power
The LiNA Xcise™ has a higher torque and runs at 1,000RPM to morcellate through the most fibrotic tissue (Largest reported uterus morcellated by LiNA Xcise was 2930gr).



Innovation

Easy and effective morcellation

- Flared cutting tube
Cutting tube on The LiNA Xcise™'s flares out to a larger diameter as tissue is pulled through, which reduces drag, provide long tissue strips and reduces tissue rotation.
- Inner bevel blade design
The LiNA Xcise™ features a large inner bevel blade design preventing the blade from going dull from inadvertent contact with the tenaculum.
- Adjustable coreguard
The coreguard on the cutting tube can be adjusted and rotated 360° according to the hand position ensuring optimal peeling effect.
- 5mm reducer cap
The reducer cap has a simple snap-on design for quick exchange and optimal use of 5mm instrument.



Safety

Full control during morcellation

- Three-step positioning
The LiNA Xcise™ has three blade positions to secure an optimal and safe morcellation; "Safety Guard" - Blade fully retracted into the cannula for patient safety "Cut 1" for surface peeling, "Cut 2" for coring

Ordering information: **LiNA Xcise™**

Email: orders@grcsurgical.com

REF.	Product	Packing
MOR-1515-6	LiNA Xcise - 15mm morcellator with obturator - single use	1 Unit, sterile

CE marked by

